

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2005 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

shower commode



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

assembly instructions

1. Place the seat, cushioned side down, on a flat surface and assemble a seat hook to each corner of the seat. Firmly tighten the screws to prevent the hooks from sliding or turning.


CAUTION:

The “L” shaped hook must be located in the corner indicated by the arrow above.



2. **CAUTION:**

Ensure that the seat stops are positioned as shown on both seat rails to prevent the seat from sliding forward. Each screw and nut should be tight enough to prevent the stop from moving without deforming the end of the seat rail, which would prevent easy folding of the shower commode chair.

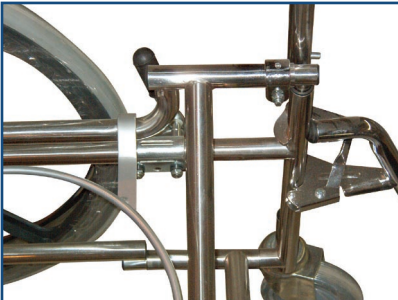

3. To install the seat, open the chair to its full width and place the seat on the seat rails by the hooks with the two “J” hooks on the seat rails with the front hook located behind the seat stop. Lower the other side of the seat until it rests on the seat rails.



CAUTION:

The curved edge of the seat must be at the rear of the chair. Lift the seat rails about 1 inch, which allows the remaining hooks to engage the seat rails, then push down on the seat, which will open the chair to its full width and lock the seat in place.

CAUTION:

Make certain that the front hooks are butted against the back of the seat stops prevent the seat from sliding forward.

4. To remove the seat, lift the seat rails about 1 inch to narrow the chair, then lift the seat out of the chair. The chair can now be folded completely.

