item # 12105KDR-2

item # 12106KDR-2

bath bench

**drive**  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

## installation instructions

WEIGHT CAPACITY 300 LBS

1. Turn bath bench upside down on flat surface
2. Insert legs in bath bench. The legs are color coded with a dot. Match the color dot to the dot on the bottom of the bench to ensure the proper leg angle. Be sure snap buttons pop out through legs completely (Figure1).
3. Turn bath bench with legs attached over so all four legs are on the floor.
4. For Item# 12105KDR-2, slide back into tubing (Figure 2). Be sure snap buttons pop out completely.

### ADJUSTING LEG EXTENSIONS:

**NOTE:** Refer to INSTALLATIONS WARNINGS in the SAFETY SUMMARY in this instruction sheet.

1. Select one (1) of the eight (8) adjustment holes located on the lower leg frame.
2. Depress the snap button and slide the adjustable leg up or down to the desired height.
3. Ensure that the snap buttons fully protrude through the same respective adjustment hole of each leg extension.
4. Repeat STEPS 2-3 for the other legs.
5. Make sure all four (4) legs are adjusted to the same height so that the bath bench sits level.

FIGURE 1

FIGURE 2

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FIGURE 2

# lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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