


## P3142-ARMU/P3142-SBMU/P3142-SPMU

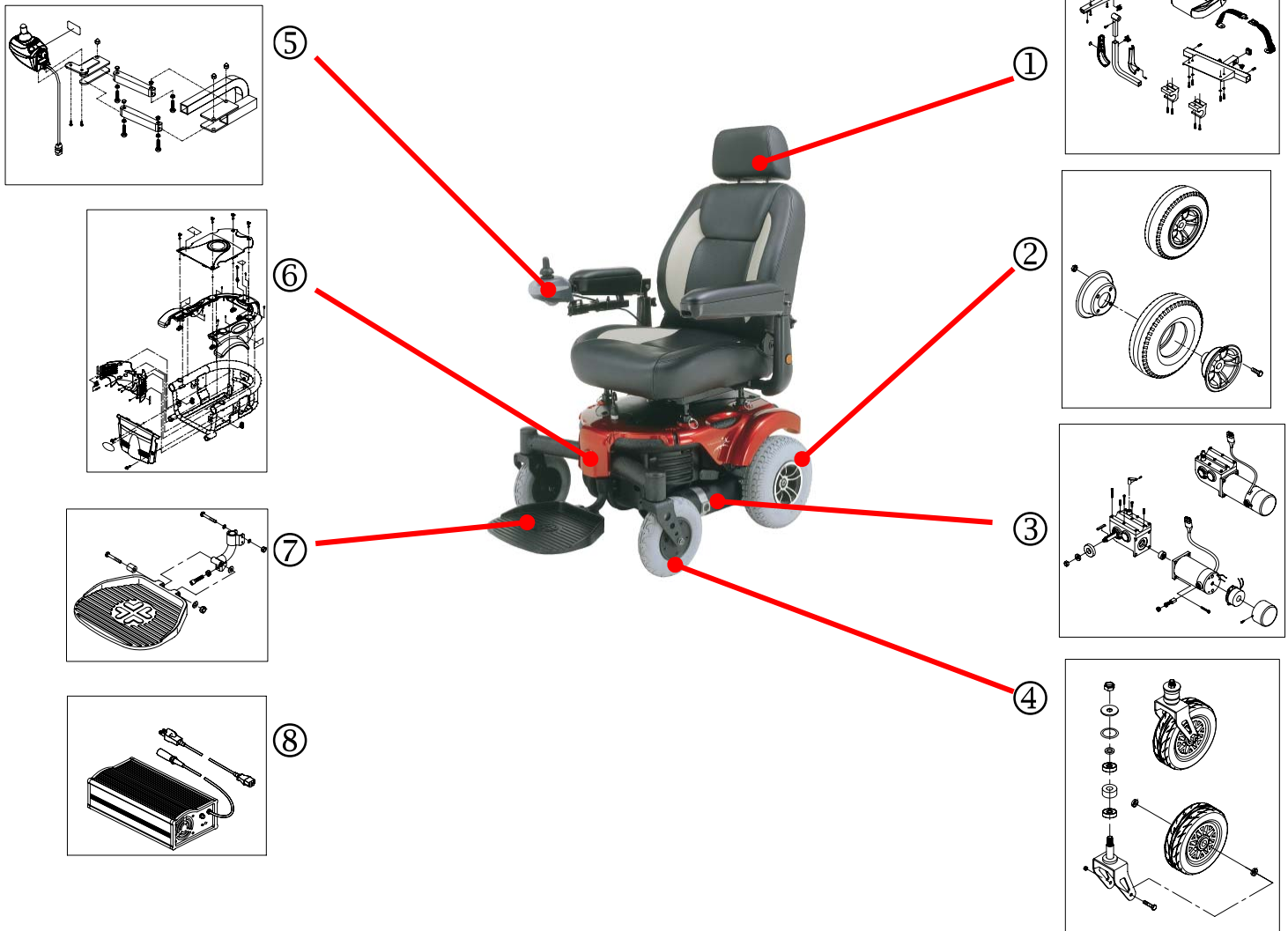

①	Seat ass'y	33100609 (CASTOR ASS'Y.,200x50 PU WHEEL, RH)
	32200160 (Captain Seat ass'y)	⑤
	32228005 (Captain Seat Only., 18wx16dx20h, Black/Gray)	33503058 (Joystick Bracket Ass'y.,Shark, Dk-Rema01, ED)
	43701072(PAD, ARMREST(LONG),BLACK)	64810012 (JOYSTICK, DY SHARK,DK-REMA01)
	43701073(PAD,ARMREST(SHORT),BLACK)	64210005 (Controller Only,Dy Sharkdk-Pma05)
②	Drive Wheel Ass'y.	⑥
	32600432 (DRIVER WHEEL ASS'Y.,260*85 PU)	30500068 (CENTER COVER ASS'Y BLACK, COLOR)
③	Motor and gearbox complete, LH and RH.	30500066 (REAR SHROUD, BLACK,P313/P314)
	62010115 (MOTOR & GEARBOX,M13KA-25, LH)	30510347 (MAIN SHROUD, RED, P313)
	62020115 (MOTOR & GEARBOX,M13KA-25, RH)	30515180 (MAIN SHROUD, BLUE, P313)
	62110070 (MOTOR ASSY., M1KA/B, LH)	30521017 (MAIN SHROUD, CHAMPAGNE, P313)
	62120070 (MOTOR ASSY., M1KA/B, RH)	⑦
	62410060 (GEARBOX ASSY., M13-25,LH)	33300341 (FOOTPLATE ASS'Y.W/MERITS LOGO)
	62420060 (GEARBOX ASSY., M13-25,RH)	⑧
④	Caster Wheel with Fork Ass'y.	Charger unit
	33100607 (CASTOR ASS'Y.,200x50 PU WHEEL, LH)	65142005 (CHARGER, HP8204-5A-OFF)
		63600008(CHARGER POWERCORD W/PLUG,US SPEC)