www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

padded raised toilet seat
with four locking clamps

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EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

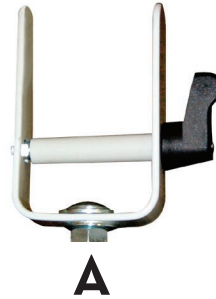
installation instructions


1. Remove Toilet Seat from carton.
2. Turn Clamp locks counter-clock wise to loosen clamp.
3. Position each clamp (A) over the toilet bowl until it rests securely on the bowl. (There is a second adjustment on the aluminum housing (B) that the clamp is attached to. To adjust, turn the entire clamp counter-clock wise until removed. There are 2 width adjustment holes for the clamp to fit in. Adjust as necessary).

NOTE: The black locks should be facing outward from the bowl.

4. Once the toilet seat is set on the bowl, tighten the black clamp lock clockwise until secure. Repeat on each clamp until all clamps are tight.
5. To remove toilet seat, reverse procedure and lift from bowl.

A

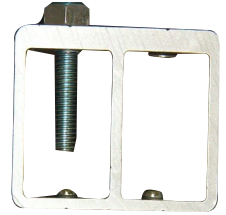B

cleaning instructions

1. Use a non-abrasive mild detergent or household cleaner with warm water.
2. Rinse seat with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product, as damage may ensue.

caution

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the clamps are tight at all times, thus securing the seat to the toilet bowl.
3. Physically challenged users should be supervised or assisted whenever using this product.

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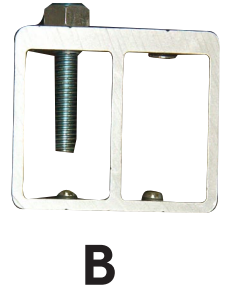
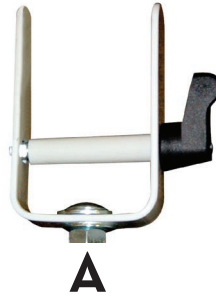
lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.