

Quick user guide

First installation:

Bathlift bellavita



1. Check stop lever and attaching the suction cups



1. Please ensure that the red seat unit locking mechanism is engaged by pushing down on the seat.




2. Screw the suction cups into the bushing and push each of the front suction cup straps into the slots on the underside of the lift.



3. When attaching the suction cups at the back of the lift, please ensure that the straps on the cups point outwards.

2. Attaching the backrest to the seat unit

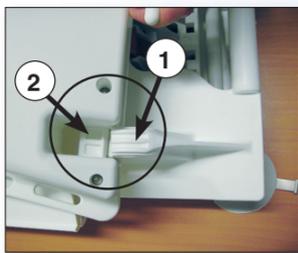

① Bearing block · ② Guide pin


1. Slide the backrest onto the hinge located on the back of the seat unit.



2. Pivot the attached backrest up until it reaches its final position.




correct

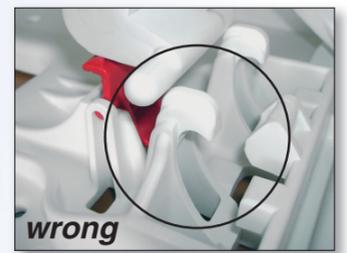


wrong

➤ **Important:**
Please ensure that the actuator end is seated properly in the base plate's ramp.


3. Charging the battery · Attaching the hand control

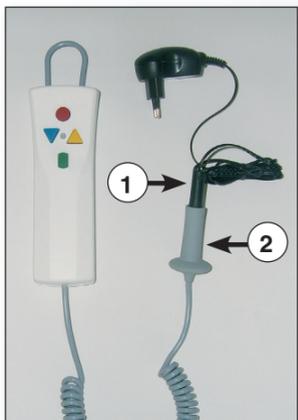

1. Place the plug ① from the charger into the cable end ② attached to the hand control. During the charging, the red light will be lit. The light changes to green when the battery is fully charged. After charging, disconnect the plug from the outlet first.

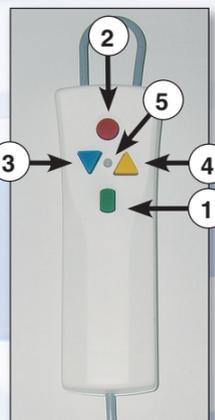

- ① Power On button, green
- ② Emergency Stop, red
- ③ Down button, blue
- ④ Up button, yellow
- ⑤ Control Light LED

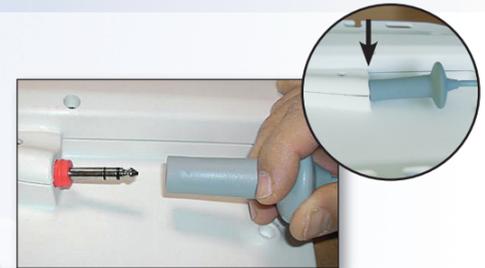

2. Hang the hand control on the backrest. Push the cable at the end of the hand controller onto the jack in the back of the backrest. Make sure that the cable is properly seated on the jack. The red ring at the jack is now unseen.

4. Removing the bathlift from the bathtub



1. Using the hand control, lower the bathlift to its lowest position where the backrest is in the fully reclined position and the lift has stop moving.



2. Move the backrest forward until it can be removed from the seat unit.



3. Ventilate both rear suction cups by pulling the straps and then pull up the underframe with the handle. The front suction cups ventilate themselves automatically.



wrong

correct

➤ **Important:**
Please ensure that the actuator is in its fully retracted position in the back rest.

USA: Drive Medical Design and Manufacturing
99 Seaview Boulevard
Port Washington, NY 11050
Phone +1 516-998-4600
Fax +1 516-998-4601
www.drivemedical.com

Great Britain:
Drive Medical Limited
Ainleys Industrial Estate,
Elland, West Yorks, HX5 9JP
Phone +44 1422 31 4488
Fax +44 1422 31 4489
www.drivemedical.co.uk

Europe:
Drive Medical GmbH & Co. KG
Leutkircher Str. 44 · 88316 Isny
Tel. +49 (0) 75 62/97 24-0
Fax +49 (0) 75 62/97 24 25
www.drivemedical.de