


1. Back Upholstery- STDS1S2630 -20i, STDS1S2632 -22i, STDS1S2634 -24i
2. Back Upholstery Screw- STDS001071
3. Rear Wheel- STDS1S1000HD
4. Rear Axle Bearing- STDS1S1012
5. Rear Axle Bolt- STDS1S1053
6. Hub Cap- STDS1055
7. Tipping Sleeve- STDS4020
8. Hand rim- STDS005008
10. Rubber Brake Tip- STDS8M4603
11. Brake Assm.- STDS4019R, right L, left
12. Front Axle Bolt- STDS8M4631
13. Front Axle Bearing- STDS1011HD
14. Front Caster Fork- STDSY3045
15. Fork Stem Bearings- STDS8M4601
16. Fork Stem Cap- STDS1051
17. Seat Upholstery- STDS2A2650 -20i, STDS1S2652 -22i, STDS1S2654 -24i
18. Seat Rail Guide- STDS1070
19. Seat Upholstery Screw- STDS001072
20. Arm Assm.- Desk STDSDDAECR or L Full STDSDFAECR or L Adj. Desk- STDSADDASPR or L Adj. Full- STDSADFASPR or L
21. Handgrip- STDS871
22. Arm Pad- Desk STDS2004BLK, Full STDS2005BLK
23. Caster- STDS1075-2
24. Slide Tube- STDS005200
25. Arm Release Button- STDS5SB
26. Footrest- STDSF-TF
27. Footplate- STDS00004A
28. Rubber Bumper- STDS1026
29. Cross Frame 20i STDS1S2620 22i STDS1S2622 24i STDS1S2624
30. Side Frame- STDS1S262R or L


Sentra EC
serial number 1Y & 1S

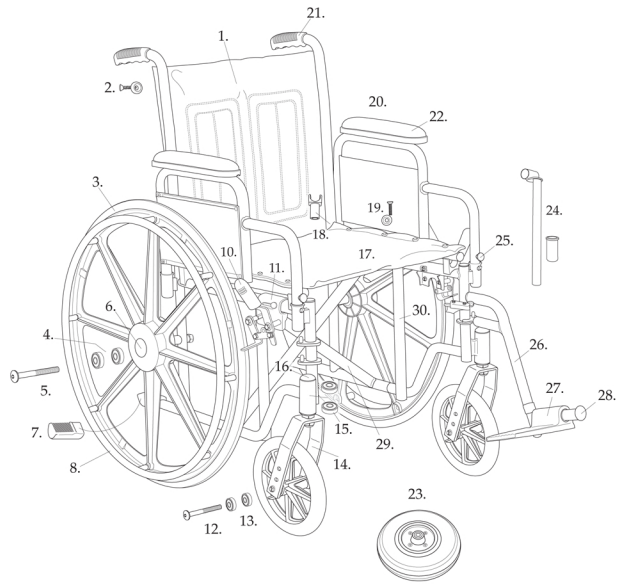

08/04/10

1. Back Upholstery- 14i STDS2A2604, 16i STDS2001BLK 18i STDS2003BLK, 20i STDS2006BLK 22i STDS2030BLK, 24i STDS2011BLK
2. Back Upholstery Screw- STDS001071
3. Rear Wheel- 16-20i STDS1000 (Chrome Rail) STDS1006 (Black Rail) 22-24i STDS1000HD (Chrome Rail)
4. Rear Axle Bearing- 16-20i Chair STDS1012 22-24i Chair STDS1012HD
5. Rear Axle Bolt- Removable Arm STDS1005 Fixed Arm STDS3020, 22&24i Chair- STDS1053
6. Hub Cap- STDS1050 for 16-20i Chairs STDS1000C for 22-24i Chairs
7. Tipping Sleeve- STDS4020
8. Hand rim- STDS004012 (Black) STDS005008 (Chrome- fixed spacers)
9. Hand Rim Screws & Spacers- STDS1073
10. Rubber Brake Tip- STDS4011
11. Brake Assm.- Removable Arm STDS4019R (right) L (left) Fixed Arm- STDS2A253R (right) L (left)
12. Front Axle Bolt- STDS1002-1
13. Front Axle Bearing- STDS1011
14. Front Caster Fork- STDS3003
15. Fork Stem Bearings- STDS1060
16. Fork Stem Cap- STDS1051
17. Seat Upholstery-14i STDS2A2634, 16i STDS2000BLK 18i STDS2002BLK, 20i STDS2008BLK 22i STDS2025BLK, 24i STDS2332BLK
18. Seat Rail Guide- STDS1070
19. Seat Upholstery Screw- STDS001072
20. Arm Assm.- (Desk) STDSDDAECR & L (Full) STDSDFAECR & L (Adj. Desk) STDSADDASPR & L (Adj. Full) STDSADFASPR & L
21. Handgrip- STDS871
22. Arm Pad- Desk STDS2004BLK Full STDS2005BLK
23. Arm Rest Panel- Desk STDS002044, Full STDS002043
24. Slide Tube- (14-20i) STDS002600 (22-24i) STDS002500
25. Arm Release Button- STDS5SB
26. Footrest- (14i) STDSF-14, (16-20i) PSTDSF-TF (22-24i) STDSF-TF
27. Footplate- FWFP-R (right) L (left)
28. Rubber Bumper- STDS1026
- 29a. Caster (14-20i) STDS1002
- b. Caster (22-24i) STDS1075
30. Cross Frame- 14i STDS2A2614, 16i STDS2A2616, 18i STDS2A2618, 20i STDS2A2620, 22i STDS2A2620, 24i STDS2A2624
31. Side Frame- (14-20i Det. Arm) STDS2A261R & (16-18i Fixed Arm) STDS2A263R & L (22-24i) STDS2A262 R & L


Sentra EC
serial number 2A

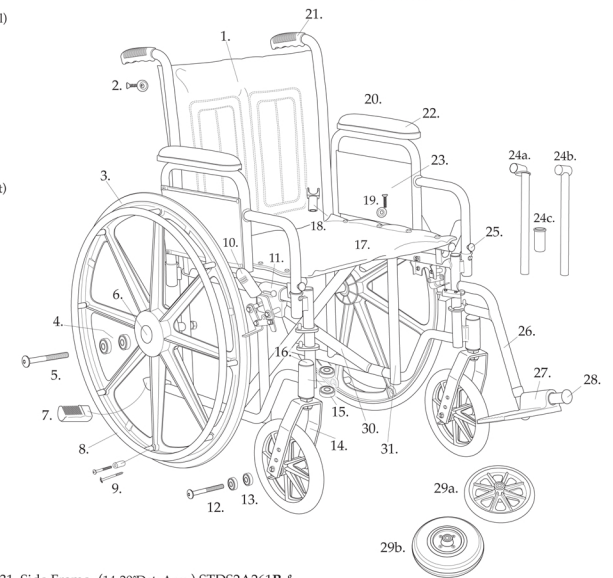

04/19/10

1. Back Upholstery- STDS1M2626 -16i STDS1M2628 -18i, STDS1M2630 -20i STDS1M2632 -22i, STDS1M2634 -24i
2. Back Upholstery Screw- STDS001071
3. Rear Wheel- STDS1006 for 16-20i STDS1S1000HD for 22-24i
4. Rear Axle Bearing- (16-20i) STDS1012 12mm ID (22-24i) STDS1S1012 16mm ID
5. Rear Axle Bolt- (16-20i) STDS1005 (22-24i) STDS1S1053
6. Hub Cap- STDS1050
7. Tipping Sleeve- STDS4020
8. Hand rim- (Black) STDS004012 (Chrome) STDS005008
9. Hand Rim Screws & Spacers- STDS1073
10. Rubber Brake Tip- STDS4011
11. Brake Assm.- STDS4019R & L
12. Front Axle Bolt- (16-20i) STDS1002-1 (22-24i) STDS8M4631
13. Front Axle Bearing- (16-20i) STDS1011 8mm (22-24i) STDS1011HD 11mm
14. Front Caster Fork- (16-20i) STDS3003 (22-24i) STDS3045
15. Fork Stem Bearings- (16-20i) STDS1060 (22-24i) STDS8M4601
16. Fork Stem Cap- STDS1051
17. Seat Upholstery- 16i STDS1M2646, 18i STDS1M2648, 20i STDS2008BLK, 22i STDS2025BLK, 24i STDS2332BLK
18. Seat Rail Guide- STDS1070
19. Seat Upholstery Screw- STDS001072
20. Arm Assm.- Desk STDSDDAECR & L Full STDSDFAECR & L Adj. Desk STDSADDASPR & L, Adj. Full STDSADFASPR & L
22. Arm Pad- Desk STDS2004BLK, Full STDS2005BLK
23. Arm Rest Panel- Desk STDS002044, Full STDS002043
24. Slide Tube- 14-20i STDS002600, 22-24i STDS005200
25. Arm Release Button- STDS5SB
26. Footrest- 16-20i PSTDSF-TF, 22-24i STDSF-TF
27. Footplate- FWFP-R & L
28. Rubber Bumper- STDS1026
- 29a. Caster 14-20i Chair- STDS1002
- b. Caster 22-24i Chair- STDS1075-2
30. Cross Frame- 16i STDS1M2616, 18i STDS1M2618, 20i STDS1M2620, 22i STDS1M2622, 24i STDS1M2624
31. Sideframe- 16-20i STDS1M261 R & L 22-24i STDS1S262 R & L

Sentra EC
serial number 1M

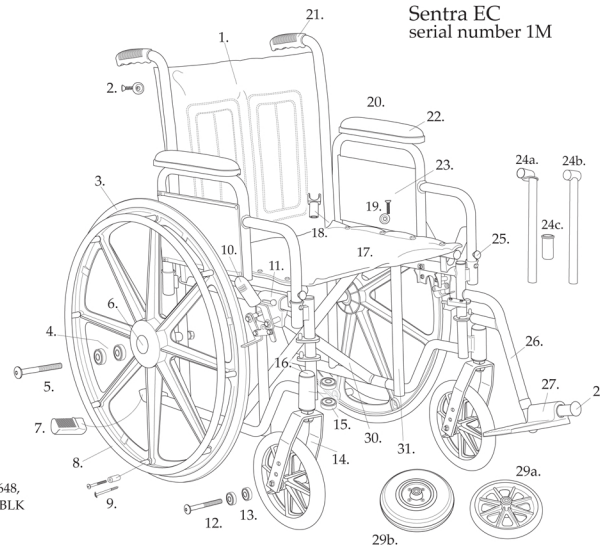

06/01/10