

How to turn off or turn on the reverse sound (LCD display screen)

1. Switch to electric mode;
2. Press and hold the "+" button for approximately 10 seconds;
3. Code "C0" or "C1" appears, then press the horn key to select "C1"
4. After selecting the code "C1", press both the "-" and "+" buttons simultaneously.
5. Code "00" or "01" will appear on the screen. Press the "-" button or "+" button to select and select to the "01" code. (Code "00" represents sound, code "01" represents no sound)
6. After selecting the "01" code, immediately press the horn button to confirm, and then reopen the controller.