

INSTRUCTIONS FOR INSTALLATION AND USE

BEFORE YOU BEGIN:

1. Maximum weight limit is 850 pounds.
2. Read and follow all instructions before use. Do not use if mat is damaged or unstable.
3. For use with wheelchairs and scooters only. Mat is intended for residential use.
4. **▲ WARNING:** Use mat only with a qualified helper and always use your lap belt.
5. **▲ WARNING:** It is important that you refer to your equipment's (wheelchair or scooter) Owner Guide for the proper degree of incline/decline and chair direction before using mat. Never exceed its recommendations.

FITTING AN ANGLED ENTRY MAT:

1. Mat can be easily trimmed to accommodate lower heights and narrower widths and, if necessary, notched to suit the doorjamb configuration.
2. If door threshold is equal to the height of the mat, use Angled Entry Mat as is.
3. If door threshold is less than the height of the mat, trim mat to the desired height as shown in **DIAGRAM 1** (serrations in the bottom of the mat can be used as a guide). **▲ WARNING:** Use caution when operating any non-manual saw as serious injury may result due to unexpected binding or pinching of rubber mat material. Always follow tool manufacturer's safety guidelines.

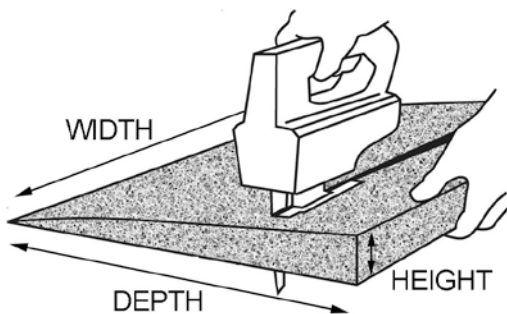


DIAGRAM 1

SECURING THE MAT: It is recommended that the mat be securely attached.

1. ATTACHING TO WOOD:

- a. Place mat in desired location and ensure that surface is free of dirt and debris.
- b. Secure mat using supplied Phillips head screws (two each). Space screws evenly (see **DIAGRAM 2** for placement) and drive until head is ¼" below the mat surface.

2. ATTACHING TO CONCRETE:

- a. Place mat in desired location and ensure that surface is free of dirt and debris.
- b. Using a ¼" diameter masonry bit, drill a hole 2" deep 1" from the edges of the corners at the taper as shown in **DIAGRAM 2**. **NOTE:** Ensure the holes are free of any residue.
- c. Drive the supplied ¼" diameter mushroom head pin anchors (two each) into the holes with a hammer until the head is seated.

CLEANING AND MAINTENANCE:

1. At all times, keep mat clear of snow, ice, dirt, leaves, and other debris that may accumulate on the surface.
2. Clean mat with mild soap and warm water. Rinse well and use extra caution when mat surface is wet.

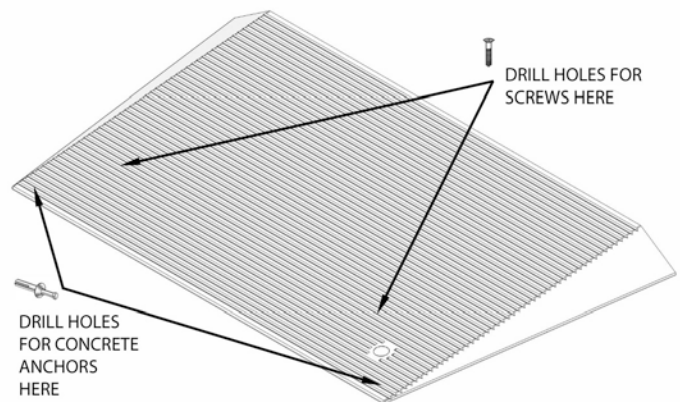


DIAGRAM 2

