

Note: This instruction sheet applies to both the VH-LG and VH-LG Mini carriers. The VH-LG is pictured for illustrative purposes.

PARTS



TOOLS



⚠ WARNING

1. Read all instructions for future reference.
2. Follow all warnings and instructions marked on the product.
3. Do not stand on the VersaHaul.
4. Do not ride any vehicle while loading the VersaHaul.
5. Always store the VersaHaul carrier in a safe place.
6. Never exceed vehicle tongue weight ratings.
7. Only use VersaHaul with Class 3 or 4 hitch. Most Class 3 hitches limit the VersaHaul to 500 lbs. total hauling capacity.
8. Improper use, storage or installation of carrier could result in death or serious injury.
9. Do not put any VersaHaul on a 5th wheel trailer or any other kind of trailer.
10. **VH-LG carrying capacity is limited to 700 lbs. max. Your vehicle suspension and hitch may limit the hauling capacity of the carrier.** For more information on vehicles and hauling capacities, visit www.versahaul.com.

LIMITED WARRANTY

The Limited Warranty on the VersaHaul Carrier.

BNK, INC warrants the carrier and related components. This warranty applies to any part found to be defective due to material and/or workmanship when inspected at the factory of BNK, Inc., Winamac, IN. BNK Inc. will, at its option, repair or replace defective parts to the original purchaser at no charge other than transportation. Sale receipt as proof of date of purchase is required, which will be returned. BNK, Inc. assumes no liability for labor charges made in performance of this warranty.

The warranty periods are as follows from date of purchase:

CARRIER FOR 1 YEAR, VOID IF: frame is broken or bent due to abuse. example: (underside scraped abusively)

ALL WARRANTIES ARE VOID IF: the carrier has been rented, used under abnormal conditions, or subjected to abuse, misuse, neglect, or improper maintenance.

All warranties should be handled at point of purchase unless other arrangements are made.

BNK, Inc. is not liable for any damage claim or liability claim, personal or otherwise resulting from operation of this carrier in any way. All transportation charges for items returned for warranty evaluation and/or replacement are to be borne by the customer.

For one (1) year from the date of purchase, BNK, Inc. will repair or replace for the original purchaser, without charge, any part or parts are manufactured by BNK, Inc. and found, upon examination at our factory in Winamac, to be defective in materials and/or workmanship.

All transportation charges on parts submitted for replacement under this warranty must be borne by the purchaser. This warranty does not apply to parts not manufactured by BNK, Inc. Their manufacturer may warrant these items. This warranty does not apply to any part which has been damaged by accident, alteration, abuse, improper adjustment, normal wear, or any other cause beyond our control.

Use of replacement parts other than genuine BNK, Inc. parts may impair the safety of your machine and void this warranty.

IMPORTANT


THIS WARRANTY IS VOID UNLESS THE PRODUCT REGISTRATION CARD IS FILLED OUT COMPLETELY AND RETURNED TO THE FACTORY AT WINAMAC, IN. WITHIN TEN (10) DAYS OF PURCHASE.

OUR WARRANTY COVERS OUR PRODUCT FOR ONE YEAR. WE ARE NOT ABLE TO EXCHANGE USED VERSAHAUL CARRIERS FOR REFUNDS OR NEW CARRIERS.

ASSEMBLING YOUR VERSAHAUL


Step 1

Insert the Tie-Down Loops into the Main Rail and fasten them with the $\frac{5}{16}$ " x $\frac{3}{4}$ " Carriage Bolts, Flat Washers, Lock Washers, and Nuts in that order (Figure 1.1). Insert the Loops from the inside of the rail. If you have the VH-LG Ramp Kit, the circled area is where the ramps will be installed (Figure 1.2). These instructions are described in detail in the separate VH-LG RK Instruction Sheet.

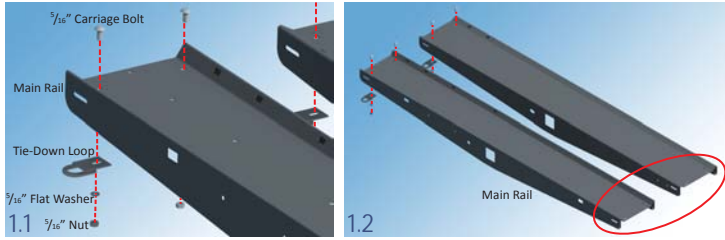


Step 2

Insert the Main Tube into the Main Rails. Make sure the main rail with tail-light mount holes and license plate holes is to the rear when attached to the vehicle. Slide the main rails to desired spacing and lock tight with the $\frac{3}{8}$ " x $1 \frac{1}{4}$ " Bolts. Make sure that the hole near the end of the Main Tube is on the bottom side with the $\frac{3}{8}$ " x $1 \frac{1}{4}$ " Bolts.

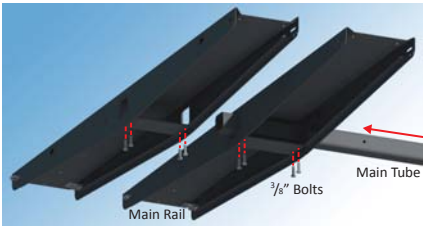


Step 3

Insert the Wheel Stop into the slots on the top of the Main Rail at the appropriate position and fasten with $\frac{5}{16}$ " Bolt, Flat Washer, Lock Washer, and Nut.

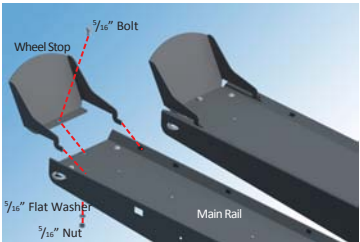

Step 4

As a safety precaution, install the $\frac{5}{16}$ " x $\frac{1}{2}$ " Bolt into the end of the Main Tube. Lock the Bolt into place with a Lock Washer and Nut.

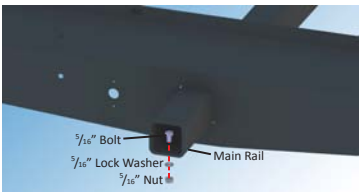

Assembled VH-LG Carrier



Remove Reflector backing and affix Reflectors as shown.



VH-LG carrier in use.

ATTACHING YOUR VERSAHAUL

Step 1

Mount the three $\frac{3}{8}$ " bolts to the included Anti-Tilt Lock Bracket. Insert bracket to the main tube as shown.



Step 2

Insert main tube into vehicle's receiver hitch and install hitch pin (not included).



Step 3

Put the bracket flush against the hitch collar. Tighten the center bolt until it fits snugly against the receiver hitch underside. Tighten the two side bolts; then re-tighten the center bolt. Make sure all bolts are tight enough such that no slack exists between the main tube and the hitch. **Note:** Road vibration may cause bolts to loosen over time. Test by rocking the carrier from the sides. Minor deformation is normal on the contact area between the main tube and the anti-tilt bracket.



Alternative Instructions

If the Anti-Tilt Lock Bracket does not fit, use the Anti-Tilt Hitch Attachment Bolt. The Anti-Tilt Hitch Attachment Bolt has a $\frac{7}{18}$ " thread size, a $\frac{3}{8}$ " Allen Head size, and a $\frac{5}{8}$ " outer head size. Slide the Main Tube of the carrier into the hitch receiver. Secure the Main Tube to the receiver with the Anti-Tilt Bolt, Washer and Nut. The Bolt must be inserted into the $\frac{7}{16}$ " hole, not the $\frac{5}{8}$ " hole. **Note:** Do not put a washer at the bolt head, but rather at the nut. The bolt head should be recessed into the receiver hole and be putting pressure on the Main Tube of the Carrier inside the receiver.



SPECIAL FEATURES



The Taillight Kit with License Plate Light is available for purchase (sold separately).



The VH-LG Ramp Option is available for purchase (sold separately).

NOTE: WHEN ATTACHING STRAPS TO TIE-DOWN LOOPS

When attaching the tie down straps, securely hook the straps from the center of the bumper bar to the eyelets located on the Tie Down Loops. Be sure to evenly tighten the straps on both sides of the ATV/Go-Kart. Go-Karts and ATVs must be strapped in on the front as well as the rear.