

# SOLAX MOBILE ARMREST – Model M-AM01-13

## Installation Instructions

1.

Locate the armrest bar underneath the seat. Then insert the end of the armrest bar as shown by the arrow.



2.

Secure the armrest as shown using the hardware that is included .



3.

Repeat steps 1 and 2 for the other side of the armrest.



# How to Fold and Unfold Armrests

## FOLDING THE ARMREST

<p><b>Original Position</b></p>  A front view of a wheelchair with its seat back upright and armrests in their standard, extended position.	<p><b>Folded Position</b></p>  A front view of the wheelchair with the seat back folded down and the armrests pushed inward and folded down.	<p><b>How to Fold Armrest</b></p>  A front view of the wheelchair with numbered arrows indicating the steps: 1. Upward arrow on the seat back, 2. Inward arrow on the armrest, 1. Upward arrow on the armrest, 2. Inward arrow on the seat back.
		<ol style="list-style-type: none"><li>1. Fold the back part of the seat down.</li><li>2. Lift the armrest up</li><li>3. Then push the armrests inward to fold down</li></ol>

## UNFOLDING THE ARMREST

<p><b>1.</b></p>  A front view of the wheelchair with the seat back upright and the armrests being pushed outward. An arrow points to the right from the left armrest, and another points to the left from the right armrest. <p>Push out the armrest and lock into place</p>	<p><b>2.</b></p>  A front view of the wheelchair with the seat back being lifted upright. An arrow points upwards from the seat back. <p>Lift the rear part of the seat up</p>
--	---