

# Solax Mobile Docking Station Instructions-Model M-DS01-13



The Solax Battery Docking Station is a portable battery charger for the Solax Lithium battery (model M-LB01-13).

## PACKAGE CONTENTS:

1. Solax Battery Docking Station
2. Power Cable

## SAFETY WARNINGS:

- Make sure no materials come between docking station contact points.
- Keep this unit away from infants and children while charging.
- Make sure the docking station is placed on a flat surface to prevent disconnect of the metal contact points while charging.

## IMPORTANT NOTES:

- Red Light indicates battery is still charging
- Green Light indicates battery is fully charged
- **Battery charge time:** 5-6 hours

## INSTRUCTIONS:

### 1.

- Connect the power cable to the power supply.
- Then connect the XLR cable to the docking station. Once this is done, plug the power cable of the docking station into a wall outlet.



### 2.

- Insert the battery with the metal contact points facing down into the docking station cradle, so that the metal contact points on the battery and docking station make contact.
- The battery indicator light will be **"RED"** if the battery is not fully charged, and **"GREEN"** if the battery is fully charged, or if the docking station and battery are not making contact.

